

## Instructions for sampling FTA Card

1



Label the card with the test kit ID (see invoice) and the animal's name.


2



Hold the pipette, capillary, or blood collection tube a few millimeters above the marked circle – do not touch the card with the pipette or capillary tip. Apply the sample volume evenly as drops onto the circle until it is completely filled.


3

 1–3 hours

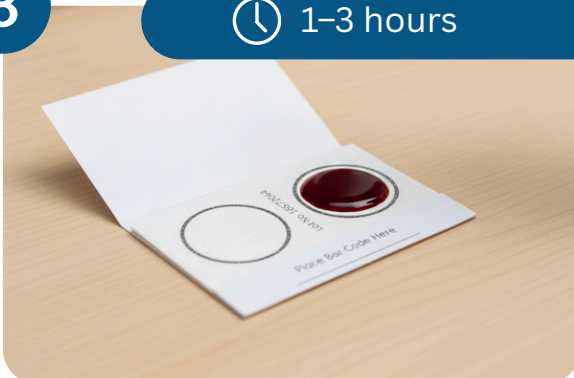

After application, let the card air-dry completely (approx. 1–3 hours, depending on humidity) – this step is crucial for sample stability and to prevent transfer or contamination.

4

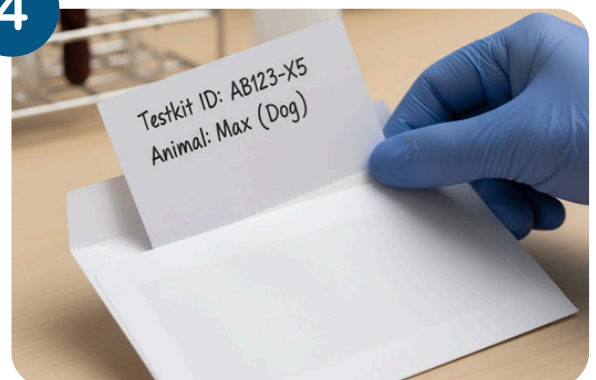

Once dried: place the cards in an envelope and ship at room temperature.

**Any questions? Our customer service will be happy to help you!**

AT +43 662 43 93 83  
DE +49 8654/68 24 430

[support@feragen.at](mailto:support@feragen.at)