

## Instructions for sampling Stool sample



Use the sampling spoon from the collection tube to take a pea-sized amount of stool. Do not overfill the tube.



Make sure that the collected sample does not come into contact with the outside of the collection tube.



Tightly close the collection tube with the cap.



Place the tightly sealed collection tube into the protective bag.



Place the protective bag along with the completed information form into the return envelope.



Send the sample on the day of collection. The sample does not need to be refrigerated for shipping.



You will receive your result by email.

## Shipping addresses

### Austria

FERAGEN GmbH  
Strubergasse 26  
5020 Salzburg  
Austria

### Germany

KUBEOS GmbH  
c/o FERAGEN  
Niedervillern 8  
83410 Laufen



Fill in all the information completely and place the form inside the return envelope.

## Kit ID:



Name:



Name:



Email:



Phone:

